

FLAGHOUSE

601 FlagHouse Drive
Hasbrouck Heights, NJ 07604-3116
www.flaghouse.com

FIBER-OPTIC LIGHT SPRAY SETS (FIBERGLASS)

US Item# 8371, CN Item# 83r71 (100 strands)

US Item# 8430, CN Item# 84r30 (200 strands)

PLEASE NOTE: A light source and color wheel are not included in the Fiber Optic Spray Sets. Please order a light source separately and refer to its separate instructions.

These fiber optic sprays have the added safety feature of crimped ends which seal the fibers inside the plastic sheaths. The fiber optic refurbishment kit is included in case accidental damage causes the glass fibers to become exposed. Repair damaged fibers immediately.

ASSEMBLY

Attach the boss to the light source. Please see instructions for light source.

SAFETY PRECAUTIONS

NOTE: Fiber Optic products are a constant source of safe tactile and visual stimulation but they should always be used under professional supervision. Please remember the following important points:

1. Though safe to stroke across the face, the strands should never be placed in the mouth. They are made from glass fibers encased in robust PVC tubes. Biting could cause these fibers to get into the mouth.
2. A refurbishment kit is included and should be used should any tails break.
3. We do not recommend the use of fiber optics for play.

SUGGESTIONS FOR USE OF FIBER OPTIC PRODUCTS

The fiber-optic light sprays change color constantly along their length to the tip where the light is brightest. The changing colors are visually stimulating and encourage focusing, yet are also calming and relaxing. The strands provide fascinating tactile stimulation, too.

Some suggested uses:

1. Attach a light source to a wall with the spray cascading down like a waterfall.
2. Stroke the strands to demonstrate that it is safe.
3. Fan out the bright tips to cast a colored light across a hand or face.
4. Spread the strands over a mat or seating and lay or sit the user on top.
5. Wrap the spray round limbs or the whole body.
6. Weave the strands in and out of a jumper. Or weave them in and out of netting which can then be draped over the user or suspended from a wall.
7. Use interactively with a switching system.

SUPPLIER'S CARE AND HANDLING RECOMMENDATIONS

Fiber optic products provide dramatic visual and tactile stimulation. However, these products should always be handled under close supervision. If misused or used carelessly they could cause injury.

The following handling points will help maximize your enjoyment of these fiber optic light spray sets.

1. Fiber optic spray should not be placed under excessive strain. Please do not allow users to pull the spray.
2. Your fiber optic spray should always be used in areas away from sharp edges.
3. The fiber optic spray should be kept away from excessive heat sources. A maximum working temperature of 167°F is recommended.
4. The use of fiber optic products is not recommended where there is a risk of accidental ingestion. Fiber optics must not be placed in the mouth.
5. If broken glass fibers accidentally contact and penetrate the skin, short-term skin irritation may result. There are no long-term effects. A practical, safe, and easy method of removal is to apply self-adhesive tape to the affected part and then gently peel it off, repeating if necessary using fresh tape. In the unlikely event that glass fibers enter the eye, immediate medical help should be sought. Should the exposed fibers fall onto the floor, vacuum the floor immediately and thoroughly. Make sure the vacuum has an appropriate filter.
6. Clean the fiber optic strands using a mild detergent and a slightly damp cloth, taking care not to wet the metal end. Dry thoroughly before reuse. Do not immerse the fiber optics in water. Fiber optics are safe when correctly used and supervised. **FlagHouse cannot accept any liability for injury or damage by the abuse or misuse of these products.**

DISCLAIMER: THIS PRODUCT MUST ALWAYS BE USED UNDER THE DIRECT SUPERVISION OF A CAREGIVER.

To ensure safety, this product must be assembled, installed, and used in strict conformity with the instructions and specifications provided. All warranties of any kind, including, but not limited to, warranties of merchantability or fitness for a particular purpose, are expressly disclaimed and no warranty of any kind is implied or expressly made. FlagHouse shall not be liable for any direct, indirect, special, incidental, or consequential damages arising or resulting from the sale, use, replacement, or repair of this product.

Please direct your questions or concerns regarding this product to the appropriate office listed below:

In the US

FlagHouse, 601 FlagHouse Drive, Hasbrouck Heights, NJ 07604-3116

Phone 800-793-7900

Fax 800-793-7922

Email: info@flaghouse.com

In Canada

FlagHouse Canada, 235 Yorkland Blvd., Suite 105, North York, Ontario M2J 4Y8

Phone 800-265-6900

Fax 800-265-6922

Email: flaghousecanada@flaghouse.com

International Customers

FlagHouse, 601 FlagHouse Drive, Hasbrouck Heights, NJ 07604-3116

Phone 201-288-7600

Fax 201-288-7887

8 AM-6 PM Eastern Time